LUXAMED®

**LUXAMED® LuxaScope Sonus
und Sonus Flex Stethoskope**

**LUXAMED® LuxaScope Sonus
and Sonus Flex Stethoscopes**

**Stéthoscopes LUXAMED® LuxaScope Sonus
et Sonus Flex**

**Estetoscopios LUXAMED® LuxaScope Sonus
y Sonus Flex**

Deutsch	2
English.....	5
Français.....	8
Español.....	11

LUXAMED®

LUXAMED GmbH & Co. KG *
Daniel-Weil-Str.3 * 89143 Blaubeuren * Germany
Telefon: +49(0)7344 92905 0 * Fax: +49(0)7344 92905 10
E-Mail: info@luxamed.de * www.luxamed.de

- DEUTSCH -

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- ENGLISH -

To ensure proper use of this LUXAMED® product, please read this user manual carefully and retain for future reference.

LUXAMED® LuxaScope Sonus and Sonus Flex Stethoscopes

Intended use:

The LUXAMED® LuxaScope Sonus stethoscopes are to be used for auscultation. Furthermore, the stethoscopes can be used to measure blood pressure according to Korotkow.

These instruments are only to be used by authorized, trained personnel.

LUXAMED LuxaScope Sonus stethoscopes are free of latex.

Use the device in accordance with its designated use and for its intended purpose only.

Warning:

Please make sure that, prior to any use, the SOFTBUDS ear tips are fixed correctly on the head piece. NEVER use without SOFTBUDS ear tips. Adjust the head piece for an ergonomic position according to your needs to enable an optimum sound quality. Make sure that the SOFTBUDS ear tips do not penetrate far into the ear canal. This might cause severe injuries.

Keep the SOFTBUDS ear tips and head piece clean to avoid deterioration of the hearing quality.

Use the stethoscopes only with Luxamed spare parts and accessories.

Application:

Stethoscopes are to be used by medical workers for auscultation of physiological sounds and noises. For the blood pressure measurement according to Korotkow stethoscopes are used to diagnose the systolic and diastolic blood pressure.

The stethoscope consists of breast piece, tube with binaural piece and fixed SOFTBUDS ear tips. Luxamed has different versions in its product range with the familiar reusable diaphragm and bell types as well as disposable versions, as described below:

Simply bending apart both binaurals to adjust the head piece its physiological conditions. To produce more pressure between SOFTBUDS ear tips and ear bend both bin-

aurals together. Be very careful to prevent overstretching.

The twin head stethoscopes consist of diaphragm and bell side which can be selected according to frequency. With one hand you fix the breast piece and with the other hand the tubing attachment.

Rotate the head of the stethoscope by 180° until it engages.

If the sound inlet on the bell side is closed, the diaphragm side or small membrane is active. The other way around, the auscultation with bell side is possible.

Different stethoscope versions:

LuxaScope Sonus: stethoscope made from stainless steel, reversible for high and low frequency sounds, available in different colours and 2 sizes (Adult/Pediatric, Pediatric/Neonatal).

LuxaScope Sonus Flex Stethoscopes: stethoscope, reversible for high and low frequency sounds, available in different colours and 1 size (Adult), exchangeable reusable diaphragm and diaphragm ring by simple clic on or pull off.

Cleaning:

The stethoscope must be cleaned with a damp cloth (use ph-neutral or slightly alkaline cleaning materials) after each use.

Disinfection:

Disinfection or disinfecting cleaning of the outer surfaces of the stethoscope simply by wipe disinfection. Use disinfectants, approved for use on medical devices made of plastic or rubber. Spray- and immersion-type disinfection is not approved. Disinfection of LuxaScope Sonus Flex stethoscopes is not absolutely necessary. The reusable diaphragm or reusable diaphragm ring have to be disinfected like mentioned above or exchanged after application.

Sterilisation:

Not approved for sterilisation.

Disposal:

This product and its accessories can be disposed of alongby with household waste.

Storage, transport and operating conditions:

Ambient temperature between -10° and +40°

Rel. humidity between 10% and 90%.

Symbol key

	Manufacturer
	Date of manufacture
	Attention to the user manual
	Caution
	In Conformity with EU guidelines
	Is a medical device

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